



ACCESSORIES

Revolution Scrubbles Apron Cleaner

683447, 683448



Scrubbles apron cleaner safely cleans your aprons of barium, blood and other common radiology related stains. Now with a clean citrus scent and a stronger more efficient formula.

Cleaning Instructions

1. APPLY Scrubbles on one section of the apron at a time and allow to stand a few minutes
2. SCRUB the area with the soft bristle brush. Do not let the solution dry before rinsing
3. RINSE the section with water and damp cloth.
4. CONTINUE steps 1-3 until the entire apron is clean.

Model	Description
683447	32 oz Bottle with Spray Cap
683448	Case of 12 32 oz bottles with Spray Caps