


# HOW TO INSTALL CASTERS ON THE BARRIERS

1


LEAN THE BARRIER TO THE SIDE, LIFTING THE LEG WITH JUST ENOUGH ROOM TO SCREW IN 2 CASTERS.

Note: Installing the casters is a job for 2 people

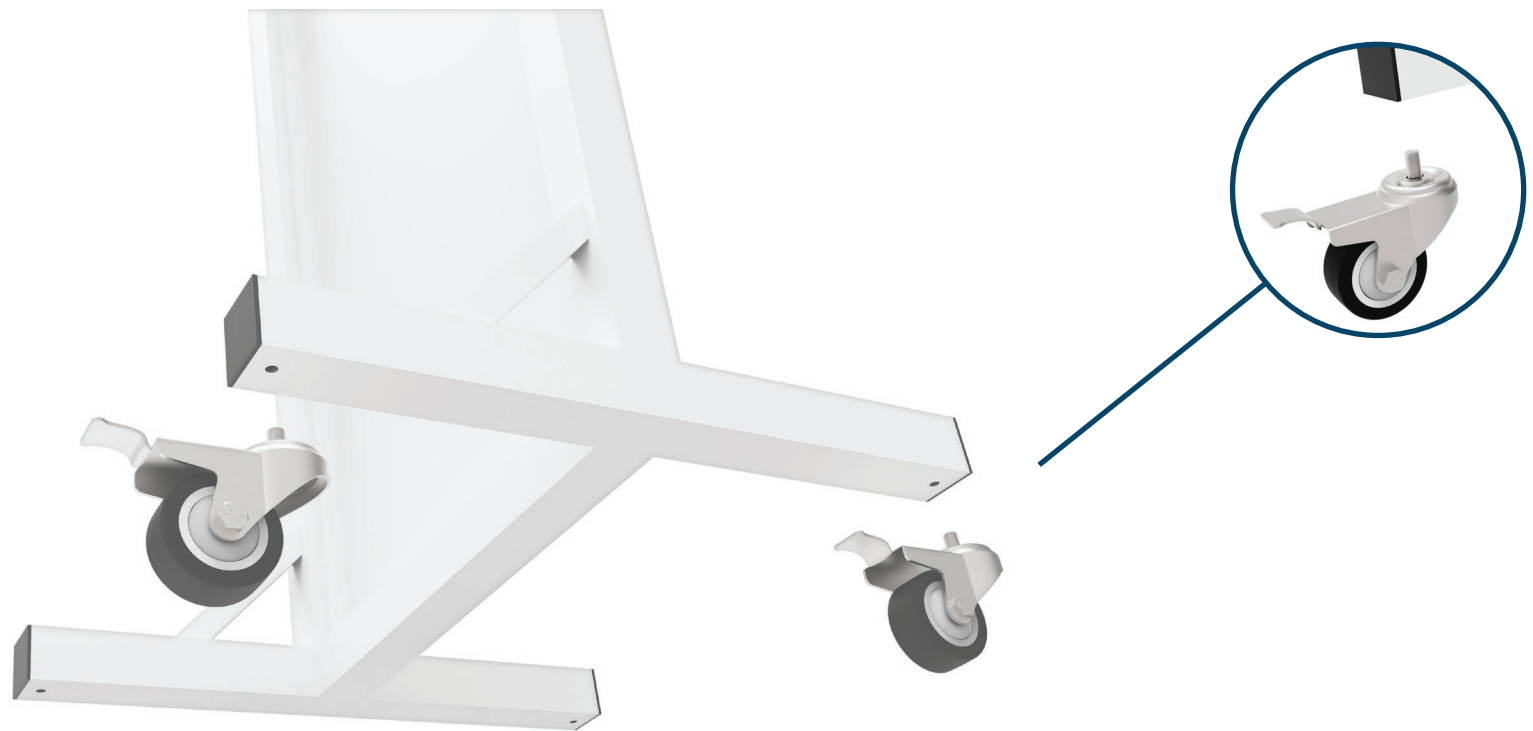

2

LOCK THE INSTALLED CASTERS BY PUSHING THE PEDALS DOWN.

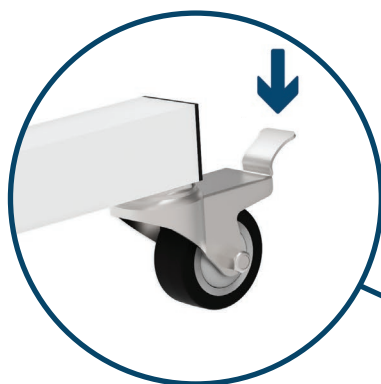

3

**TILT THE BARRIER TO THE OPPOSITE SIDE TO INSTALL THE OTHER 2 CASTERS.**

**REPEAT STEPS 1 AND 2**



4

RETURN TO UPRIGHT POSITION. UNLOCK THE CASTERS TO MOVE THE BARRIER.



The caster installation is now complete!