



Instructions to assemble RAR-MAR 36 & RAR-MAR48

PARTS LIST

1, 2 : Wheeled Base Supports 6, 7 : Vertical Rack Supports
 3, 4 : Base Mounting Plates 8 : Rack Cross Piece & Hangers
 5 : Base Cross Piece In Bag : 8 - 5/16 x 2 " Bolts

Assembly Instructions

- 1) Attach Base Mounting plates(3,4) to Wheeled Base Supports(1,2) using six of the 5/16" x 2 " Bolts and Nuts. Three on each side.
- 2) Remove the 3/8" x 2" Bolts from the Base Cross piece and use to mount in between the Base Supports.
- 3) Mount the Vertical Rack Supports (6,7) to the Base Mounting Plates(3,4) on each side using remaining 5/16" x 2 " Bolts.
- 4) Remove the 3/8" x 2" Bolts from the Rack Cross Piece (8) and attach one side to one of the Vertical Rack Supports(6,7).
- 5) Slide the 5 Hangers on the Rack Cross Piece (8) and attach to other Vertical Support using remaining 3/8" x 2" Bolt

© 2015 Phillips Safety Products Middlesex, NJ 732) 356-1493